Medisorb H

EN

Medisorb H
Hydrocolloid dressing, sterile

Product description

Medisorb H is a sterile adhesive dressing consisting of a hydrocolloid layer which does not contain gelatine, and thin semi-permeable polyurethane film providing barrier to bacteria, viruses and water.

Action

The hydrocolloid layer coming into contact with wound exudates forms a cohesive gel that absorbs exudate and provides adequate moisture conducive to wound healing. The moist environment supports the process of granulation and epithelisation. The gel formation prevents the adhesion of the dressing to the wound, so that its removal does not cause pain, and does not damage new cells.

Indications

Medisorb H is a suitable dressing for the moist wound healing and can be used in the treatment of wounds with moderate exudate, such as:

- 2nd degree burns,
- leg ulcers,
- bedsores,
- chronic slow-healing wounds and abrasions.

How to use

- Before applying the dressing, clean the wound with normal saline or a Ringer's solution.
- Dry the skin surrounding the wound.
- Select the appropriate size of the Medisorb H dressing so that it covers the wound surface and 2-3 cm of the skin around it. When covering a large surface area several overlapping dressings can be applied.
- Remove the release paper. Apply Medisorb H directly onto the wound surface and gently press the edges of the dressing.
- No extra dressing is needed to fix Medisorb H.
- The dressing should be changed when the hydrocolloid surface turns into a gel. The dressing change frequency depends on the amount of discharge. Heavy exuding wounds may require daily dressing changes. In case of low exuding or epithelialising wounds the dressing can remain on the wound for up to 7 days.

Additional comments

- the lack gelatine minimises the occurrence of allergies and eliminates unpleasant odour,
- the semi-permeable polyurethane layer protects the wound against water and bacteria,
- high absorbent properties of the dressing eliminate excess exudates, thus prolonging time between dressing changes,
- the dressing speeds up the healing process by providing an ideal environment for the formation of granulation tissue,
- a slightly transparent structure enables to control wounds.

Contraindications

Medisorb H should not be used:

- on 3rd degree burns,
- on exposed muscles, bones, tendons or infected wounds.

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