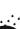
# Medisorb F

# Medisorb F

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## EN

**MEDISORB F**  
Polyurethane transparent film dressing, sterile

**PRODUCT DESCRIPTION**  
Medisorb F wound dressing is a sterile transparent dressing, which consists of a polyurethane membrane, coated with an acrylic adhesive.

**ACTIVITY**  
The dressing is impermeable to micro-organisms however it is permeable to water vapour and oxygen. Medisorb F maintains a moist environment, which prevents scab formation and is conducive to natural healing. Medisorb F allows visual inspection of the wound throughout the healing process.

**INDICATIONS**  
Medisorb F wound dressing is indicated for the treatment of:

- minor burns,
- donor sites,
- stage I and II pressure areas,
- post-operative wounds,
- abrasions and lacerations.

Medisorb F is also used as protection against skin breakdown due to friction or continuous exposure to moisture.

**APPLICATION**  
1) Thoroughly cleanse the wound with sterile saline or Ringer solution.  
2) Ensure that the surrounding skin area is clean and dry.  
3) Choose the appropriate size of the dressing so that at least 2 to 3 cm perimeters around the wound site for attachment.  
4) Remove the printed release paper from the central part of the dressing and holding by the edges place it on the wound.  
5) Remove the release paper from the edges and stick the rest of the dressing. Gently apply pressure to the dressing as it attaches to the wound.  
6) Once the dressing is secured remove the plastic carrier by peeling the tabs in the direction of the arrows.  
7) The dressing may be left in place for up to seven days before another dressing is required. The dressing should be removed immediately if there are any clinical signs of infection. Resume the use of Medisorb F when normal healing conditions are present.

**PRECAUTIONS**

- For topical use only.

**CONTRAINDICATIONS**  
Medisorb F should not be used:

- for third degree burns,
- for deep cavity wounds,
- in case of infection with inflammatory signs (temperature, oedema, redness) remove the dressing and contact proper medical authority; resume the use of Medisorb F when normal healing conditions are present,
- if package is damaged or opened.

**GENERAL INFORMATION**  
Disposable product. Sterile. Sterilised using irradiation. Do not resterilise. Do not use if the package is damaged. Do not use after the expiry date. Protect against sunlight and moisture. Re-use of the product carries a very high risk of infection of the patient and is unacceptable. Store away from heat sources, in conditions that protect against mechanical damage and chemical contamination.

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